

Minuet Watch

What is the Minuet Watch?

The Minuet Watch is a high quality watch that incorporates a personal trigger compatible with Tunstall's range of Lifeline home units and other Tunstall telecare enabled systems. It operates on the European 869MHz frequency and provides the wearer with increased reassurance that they have a means to contact help when and if they should need it.

The Swiss designed watch is waterproof to IP67 standard to provide protection from brief submergence in water and also incorporates an LED to provide the user with visual reassurance that the alarm button has been pressed.

How does it work?

The Minuet Watch can be used in exactly the same way as a standard personal trigger. For example, when the user requires assistance all they need to do is press the alarm button on the watch face. The Minuet Watch will then send a radio signal to the Lifeline home unit which in turn will then raise a call to the monitoring centre.

The Minuet can be worn as a watch around the wrist using the supplied wrist strap or around the neck using the supplied neckcord and holder.

Who is it for?

The watch is designed for use by all people who would benefit from carrying a personal trigger including those with physical, health or age-related conditions. The watch is also useful for younger telecare users who prefer a more modern design to the traditional personal trigger. It may also be useful for instances where the wearing of a personal trigger needs to be more covert e.g. domestic violence. It also uses a standard sized watch strap enabling the user to personalise their watch with a wide range of straps available from high street retailers.



Why is it needed?

The Minuet Watch has been particularly developed to help encourage telecare users to wear their personal triggers throughout the day. By combining an alarm button with a high quality watch, users are more likely to wear it and as a result will be provided with additional protection as their ability to raise an alarm call is increased.



Features and benefits

- **High quality** - Swiss designed watch and ultra reliable alarm operation
- **Lightweight** - making it comfortable to wear
- **Easy to change wearing options** - the watch face can be simply swapped between the wrist strap and the neck cord to suit the user's preferred wearing option (see below)
- **Customisable straps** - the watch strap is standard 18mm sized allowing it to be interchanged with a wide range of personalised straps available from high street jewellers and watch repairers
- **Alarm button** - integrated into the modern design of a watch
- **LED indication** - informs the user that the alarm button has been pressed and when the battery is low
- **Auto low battery** - the watch reports to the monitoring centre when the battery is low
- **Waterproof** - to IP67 standard covering brief submergence in water
- **Compatibility** - compatible with Tunstall Lifeline Connect/Connect+/400/4000+ home units and other Tunstall telecare enabled systems

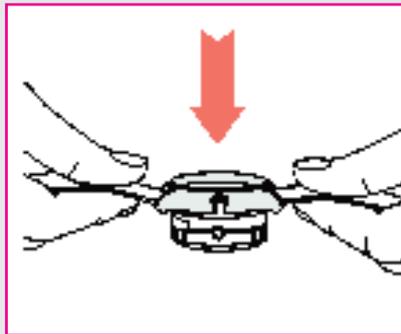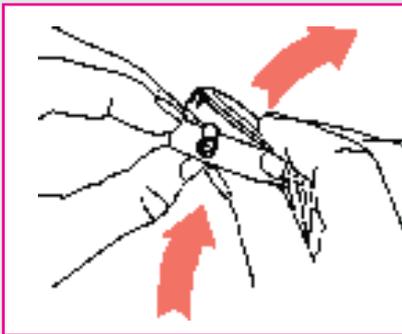

Technical details

Radio Frequency:	869.2125MHz
Radio Range:	Up to 50m (in building), 200m (line of sight)
Battery Type:	Radio Transmitter: CR2032
Battery Life:	Watch: 5 years (non replaceable) Radio Transmitter: 3 years (replaceable)
Dimensions:	46 x 38 x 13 mm (L x W x D)
Watch strap size:	18mm (standard size)
Weight:	24g
Operating temperature:	-10°C to +55°C
Watch accuracy:	-10/+20 sec / month (at 25°C)
Waterproof:	IP67
CE:	Compliant
Standards:	EN 50134-2
Part Number	67605/06

Our policy of continual development means that product specification and appearance may change without notice. Tunstall does not accept responsibility for any errors and omissions contained within this document.

© 2012 Tunstall Group Ltd.
® TUNSTALL is a registered trademark.

Tunstall Healthcare (UK) Ltd,
Whitley Lodge, Whitley,
Yorkshire DN14 0HR

Tel: 01977 661234
Fax: 01977 662570

Email: enquiries@tunstall.com
tunstall.com



Tunstall